



Instructions for Use
Dry Heat Chemical Indicator Strips
Item #: 3470-415

Dry Heat sterilizers for use with static air or forced air dry heat processes up to 160°C. Used for Dry Heat Sterilizers. Color indicator changes from lavender to green indicating the sterilization process has been completed.

During exposure to dry heat sterilization at 160°C, the indicator color will change from lavender to reference color or equivalent.

Can be written on and kept as a permanent record.

Shelf Life/Expiry Date

Products stored in recommended conditions have a shelf life of 3 years from date of manufacture.

Manufacture, Expiry Date and LOT number are printed on the product box.

Storage Conditions

Strips should be stored in a dry condition at room temperature and protected from direct light, moisture and excess humidity.

Prevent moisture, do not contact with acid and alkaline substances.