

















Product Instruction

Date of issue: 2025-03-25

Trademark of Distributor			
Name of product	Nonsterile cotton tipped applicators		
Specification	3" and 6"		
Applicable item codes	4556-385 / 4556-394 / 1015-867 / 1015-876		
LOT	Refer to packaging	Production / Expiration Date	Refer to packaging
FDA registration number	9680887		
Device listing number	D153430		
Intended purpose	They are primarily used to transfer and apply medical agents to wounds, or absorption of limited exudates.		
Contraindication	Product is prohibited for those allergic to cotton fiber.		
 Caution	This product is disposable. Please use a new cotton tipped applicator each time. This product is not sterilized before delivery and is not provided aseptically.		
Use Method	Read the label and instructions for use carefully before use. Ensure the package was not damaged and the product is within its validity date. Put the sealed bag into steam sterilizer for sterilization. After a cycle of sterilization, check the color changed on the bag and make sure it completely showed as indicators color change standards (Blue indicator on bag will turn black / gray when processed by steam at 134°C/4min). Wash hands thoroughly with soap and water or clean hands with alcohol-based sanitizer. Open the bag and take product out of the bag. Avoid touching the cotton tip. Use the cotton tipped applicator to transfer and apply medical agents to wounds, or absorption of limited exudates.		
Storage conditions	Products should be stored in a sealed, moisture-proof, ventilated and dry place. Relative Humidity: Less than 80%; Temperature: Less than 40°C.		
Shelf-life	The product shelf life is 5 years.		
Disposal	Please dispose of the product after use to comply with local regulation.		

[Definitions of Signs]

Signs	Description	Signs	Description	Signs	Description
	Trademark of Distributor		Batch number		Date of Manufacture
	Medical device		Consult instructions for use		Country of manufacturer
	Use-by date		Do not use it if the package is damaged		Caution
	Do not reuse		Keep away from sunlight		Unique Device Identifier
	Keep dry		Manufacturer	/	/