

**BencoDental**



## **VISION XR AUTOMATIC FIXER/ DEVELOPER**

### **INSTRUCTIONS FOR USE**

#### **GENERAL INFORMATION**

Specifically designed for all types and brands of dental X-ray film, as well as brands of automatic film processors. Premixed Developer for Automatic dental processors, DO NOT DILUTE.

- Each chemical is color-coded - developer is a reddish (light rose) color: fixer is blue.
- Chemicals are odorless.
- Oxidation resistant.
- High capacity.

#### **INSTRUCTION FOR USE**

- Take off the cap.
- Pour each 1.3 gallon jug into it's respective chamber (i.e. Develop to Developer/Fixer into Fixer).
- Fill the chambers with the appropriate chemicals.

#### **REPLENISHMENTS**

- Automatic Replenishment - Follow manufacturer's recommendations. NO STARTER REQUIRED. DO NOT DILUTE.
- For best results, solutions should be replenished daily.
- The developer is a reddish (light rose) color and the fixer is blue. The developer red will go clear if contaminated with fixer.

#### **SPILL**

A spill can be neutralized with baking soda (sodium bicarbonate).

#### **WARNING**

Keep out of reach of children and animals. Use caution when handling any chemicals. If contact with skin or eyes occurs, wash thoroughly for 15 minutes. If eye irritation occurs or persists, seek medical attention.

Distributed by

**BencoDental™**

800.GO.BENCO • BENCO.COM  
295 CenterPoint Blvd, Pittston, PA 18640