



INSTRUCTIONS FOR USE

Description:

An aqueous-based universal cleanser suitable for non-abrasive cleaning of the intaglio surface of prosthetic restorations after intra-oral try-in.

Indications for Use:

Cleaning of ceramic and metal substrates prior to bonding or cementing in place. Significantly removes phosphate contamination from zirconia surfaces.

General Instructions (Bottle):

1. After intra-oral try-in and determining the restoration is ready to deliver, thoroughly rinse with water to remove any contamination, and dry with oil-free air.
2. Dispense several drops of the eon universal cleanser into a well. Using an applicator brush, or micro applicator, liberally apply to the entire intaglio surface of the restoration using a gentle brushing motion.
3. Allow to dwell for 20 seconds, then thoroughly rinse with water and dry with oil-free air.
4. Prime the bonding surface of the restoration with a suitable bonding agent. Make sure to observe the instructions for use of the bonding agent and the luting material used.

Target Patient Groups:

Patients undergoing restorative procedures using ceramic and metal substrates.

Intended Users:

For professional dental use only.

Storage:

Store between 2°C/36°F and 30°C/86°F. Keep container tightly closed. Keep only in the original container in a cool well-ventilated place. Keep container closed when not in use. Do not store with strong acids and oxidants. Do not use past the expiration date.

Contraindications:

Contraindicated for in vivo use; only intended for extraoral use.

Precautions:

1. Only intended to be used by dental professionals.
2. Ensure adequate ventilation during use.
3. Wear protective eye wear and gloves
4. Avoid contact with eyes, skin, and soft tissue.
5. Application tips or brushes are single use only. Reuse of these items may result in cross-contamination between patients.
6. To avoid cross contamination, recap and wipe product with a suitable medical grade disinfectant between uses.
7. Wash hands and other exposed areas with mild soap and water before eating, drinking or smoking and when leaving work. Wash contaminated clothing before reuse. Do not eat, drink or smoke when using this product. Always wash hands after handling the product. Handle in accordance with good industrial hygiene and safety procedures.
8. Keep out of reach of children

WARNING:

Causes skin irritation. Causes serious eye irritation. May cause respiratory irritation. Avoid breathing dust/fume/gas/mist/vapors/spray. Wash skin thoroughly after handling. Use in a well-ventilated area. Wear protective gloves/eye protection/face protection. Disposal should be carried out in compliance with federal, state and local regulations.

FIRST AID:

IF ON SKIN: Wash with plenty of soap and water.

IF INHALED: Move to fresh air and keep at rest in a position comfortable for breathing.




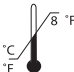


IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do so. Continue rinsing. If irritations persist, seek medical advice/attention.

IF INGESTED: Rinse mouth. Do not induce vomiting unless medical advice is given from a professional. Call a poison control center if feeling unwell and/or if irritation persists.

Note:

Visit bencodental.com for SDS.

Symbol	Title of Symbol	Standard Containing the Symbol	Function/Description per Standard	Manufacturer Interpretation
	Distributor	ISO 7000 & ISO 15223-1	To identify the distributor of a product.	Distributor
	Manufacturer	ISO 7000 & ISO 15223-1	Indicates the medical device manufacturer	Manufacturer
	Catalog Number	ISO 7000 & ISO 15223-1	To identify the manufacturer's catalog number, for example on a medical device or the corresponding packaging. The catalog number shall be placed adjacent to the symbol.	Catalog Number
	Lot Number	ISO 7000 & ISO 15223-1	Indicates the manufacturer's batch code so that the batch or lot can be identified	Lot Number/ Batch code

	Use by Date/ Expiration Date	ISO 7000 & ISO 15223-1	Indicates the date after which the medical device is not to be used	Use by Date/ Expiration Date
	Operator's manual; Operating Instructions	ISO 7000	To identify the location where the operator's manual is stored or to identify information that relates to the operating instructions. To indicate that the operating instructions should be considered when operating the device or control close to where the symbol is placed.	Follow the operating instructions for use
	Do not Use if Package is Damaged	ISO 7000 & ISO 15223-1	Indicates a medical device that should not be used if the package has been damaged or opened and that the user should consult the instructions for use for additional information.	Do not use if seal or packaging is compromised
Rx ONLY	Prescription Only	FDA Title 21	Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner	Device is for prescription use only by a medical/dental professional
	Temperature Limits	ISO 7000 & ISO 15223-1	Indicates the temperature Limits to which the medical Device can be safely exposed	Product has special storage constraints. Store product between the temperature values indicated
	Caution	ISO 7000 & ISO 15223-1	To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.	Consult the instructions for use for important cautionary information
	Exclamation Mark	Global Harmonized System (GHS) for classification and labeling of chemicals	Irritant-irritates the skin or eyes; Skin sensitize-which is an allergic response following skin contact; Acute toxicity- which may be fatal or cause organ damage from a single short-term exposure; Narcotic effects like drowsiness, lack of coordination, and dizziness; and Respiratory tract irritation.	Product is considered an irritant and/or sensitize. Handle with care and wear appropriate personal protective equipment (PPE).



Distributed by

BencoDental

800.GO.BENCO • BENCO.COM
295 CenterPoint Blvd, Pittston, PA 18640

MADE IN USA