



## Instructions for Use (IFU)

### Earloop Facemasks

natural extensions® | Benco Dental™ | ValuLine®

#### 1. UNPACKING

---

- Perform proper hand hygiene before handling the mask.
- Remove a new mask from its packaging.
- Inspect the mask for damage, defects, or contamination.
- Do not use if the mask is damaged or contaminated.

#### 2. DONNING THE MASK

---

- Hold the mask by the earloops with the outer side facing outward.
- Place the mask over your nose and mouth.
- Adjust the nosepiece to fit securely over the bridge of your nose.
- Pull the mask down to fully cover your chin.

#### 3. SECURING THE MASK

---

- Place the earloops around your ears.
- Ensure the mask fits snugly against your face with no gaps along the edges.

#### 4. DURING USE

---

- Avoid touching the mask while wearing it.
- If the mask is touched, perform hand hygiene immediately.
- Replace the mask if it becomes damp, soiled, damaged, or difficult to breathe through.

#### 5. REMOVAL AND DISPOSAL

---

- Perform hand hygiene before removing the mask.
- Remove the mask by handling the earloops only.
- Lean forward slightly and pull the mask away from your face.
- Dispose of the mask in an appropriate waste container in accordance with local regulations.

**! CAUTION:** Single-use only. Do not reuse. Dispose of the mask after each use. Keep out of reach of children unless supervised.

## STORAGE

---

- Store in a cool, dry place away from direct sunlight.
- Avoid exposure to excessive heat (above 40°C / 104°F).

## WARNINGS AND PRECAUTIONS

---

- Single-use only. Do not reuse.
- Dispose of the mask after each use.
- Keep out of reach of children unless supervised.

**Questions? Contact Benco Dental Customer Care**  
1-800-GO-BENCO (1-800-462-3626) | [benco.com](https://www.benco.com)

Distributed by

***BencoDental***<sup>™</sup>

800.GO.BENCO • BENCO.COM  
295 CenterPoint Blvd, Pittston, PA 18640