



natural extensions® gen2 White Sensitive Mask ASTM Level 3 Box of 50 - [6324-685]

natural extensions® gen2 Blue Sensitive Mask ASTM Level 3 Box of 50 - [6324-694]

natural extensions® gen2 Black Sensitive Mask ASTM Level 3 Box of 50 - [6799-100]

INDICATIONS FOR USE:

The disposable earloop face mask is intended to be worn to protect both the patient and healthcare personnel from transfer of microorganisms, bodily fluids and particulate material. The disposable earloop face mask is intended for use in infection control practices to reduce the potential exposure to blood and bodily fluids. The mask is a single use, disposable device(s), provided non-sterile.

CONTRAINDICATIONS:

Masks are single use and should not be re-used.
They should not be worn near flammable anesthesia gases.

INSTRUCTIONS FOR USE:

1. Unpack the Mask: Open the inner packaging and remove the mask.
2. Proper Orientation: When putting on the mask, ensure that the earloop side is always on the outside.
3. Positioning the Mask: Attach the mask to your face and nose horizontally with the natural extension logo facing upwards.
4. Nose Clip Adjustments: Place the fingertips of both hands on the nose clip, starting from the middle position. Press inward and gradually move to the sides to shape the nose clip to fit your nose.
5. Chin Clip Securement: Pinch the center of the chin clip to secure it against your chin. Then move your fingers to the sides to ensure the chin band fits the shape of your chin.
6. Earloop Placement: Pull the earloops over your ears while wearing the mask.
7. Extend the Mask: Unfold any pleated creases and extend the mask for proper coverage.

Distributed by

BencoDental™

800.GO.BENCO • BENCO.COM
295 CenterPoint Blvd, Pittston, PA 18640