

Z3™ Waterline Treatment Tablet

Instructions:

1. Place tablet or tablets in an empty dental unit water bottle every time you refill the water. Z3 is intended for use with potable water (including tap or distilled).
 - For 750mL Tablets
 - 700-750mL – 1 Tablet
 - 1.5L – 2 Tablets
 - 2L – 3 Tablets
 - For 2L Tablets
 - 2L – 1 Tablet
 - 4L – 2 Tablets
2. Fill the bottle with the desired volume of water and connect it to the dental unit. Tablet dissolves in a few minutes.
3. Z3 may be left in the waterlines overnight and for extended periods of non-use. Z3 has been shown to be effective for up to 28 days. Use water solution until it is expended; or up to 28 days. It is not necessary to purge lines at the end of each day.

Clinical monitoring of water quality can ensure that procedures are performed correctly. Test water quality quarterly to establish the effectiveness of your maintenance program. If water quality results approach or exceed requirements, repeat Initial Treatment.