

BencoDental

GIBSON HEALTHCARE®

Wound Dressing Gel

10 mL syringe

BencoDental
GIBSON HEALTHCARE®
Wound Dressing Gel

**INSTRUCTIONS
FOR USE**

10 mL syringe
READ CAREFULLY BEFORE USE

CAUTIONS

Do NOT use Gibson Healthcare Wound Dressing Gel if you are allergic to any ingredients. Consult Your Doctor or Dentist:

- If an allergic reaction occurs
- If you have allergies to medicines, foods, or other substances
- If you are pregnant or breast feeding
- If you are taking any prescription or nonprescription medicine, herbal preparation, or dietary supplement

Contraindications:

The administration of Gibson Healthcare Wound Dressing Gel is contraindicated in any patient with a known or suspected hypersensitivity to any of its ingredients.

Contains cinnamon

INGREDIENTS:

Purified Water, Xylitol, Acemannan (an Aloe Polysaccharide), Guar Gum, Sodium Bicarbonate, Potassium Sorbate, and Essential Oils of Cinnamon, Clove, & Thyme

PRESCRIBING INFORMATION:

INDICATIONS

- Gibson Healthcare Wound Dressing Gel manages the pain of oral wounds, mouth sores, injuries and ulcers of the oral mucosa.
- It adheres to oral tissue and forms a protective barrier between the wound and further irritation & contamination. Gibson Healthcare Wound Dressing Gel also maintains a moist wound environment.

DESCRIPTION

Gibson Healthcare Wound Dressing Gel is a gel for the relief and management of oral pain. Gibson Healthcare Wound Dressing Gel is packaged in syringes for precise application and will be provided non-sterile and is not intended to be sterilized before use. Because Gibson Healthcare Wound Dressing Gel is to be administered several times per day; it will primarily be used at the home, office, or other non-clinical settings. However, Gibson Healthcare may also be used in clinical or healthcare facility settings where patients are receiving services.

PURPOSE

The goal of administering Gibson Healthcare Wound Dressing Gel is to manage the pain of oral wounds, mouth sores, injuries and ulcers of the oral mucosa. It adheres to oral tissue and forms a protective barrier between the wound and further irritation & contamination. It also maintains a moist wound environment.

DOSAGE AND ADMINISTRATION

Remove the syringe cap and attach the curved tip. Use the curved tip to apply the gel directly onto the affected site in the mouth. Apply enough to cover the lesion or wound site and allow to remain in place as long as possible without disturbing with the tongue or finger. Gibson Healthcare Wound Dressing Gel will slowly dissolve. Apply again as often as needed for pain.

HOW SUPPLIED

Gibson Healthcare Wound Dressing Gel is supplied as a gel in a syringe. Store at room temperature conditions 15°C to 30°C (59° to 86°F). Do not refrigerate.

INSTRUCTIONS FOR PATIENTS:

PROCEDURE FOR ADMINISTRATION

Gibson Healthcare Wound Dressing Gel

1. Remove the syringe cap by untwisting
2. Twist the curved tip on the syringe
3. Use the curved tip or finger to apply the gel directly onto the affected site in the mouth.
4. Apply enough to cover the lesion or wound site and allow to remain in place as long as possible without disturbing with the tongue or finger. Gibson Healthcare Gel will slowly dissolve.
5. Replace syringe cap when not in use.
6. Apply again as often as needed for pain

If no improvement is noticed after 10-14 days, consult your clinician.

Store Gibson Healthcare Wound Dressing Gel at room temperature.

Do not refrigerate.

Keep out of the reach of children.

CAUTION:

Federal law restricts this device to sale by or on the order of a physician or dentist. In case of serious incident using Gibson Healthcare Wound Dressing Gel, please get medical help and report to Benco Dental or appropriate agencies.

REF IFUGWD1



15°C
(59°F)

30°C
(86°F)

Distributed by

BencoDental™

800.GO.BENCO • BENCO.COM
295 CenterPoint Blvd, Pittston, PA 18640