

ValuLine®

bite registration

Instructions for Use

This document contains the most current Instructions for Use. Please read and retain.

DESCRIPTION

The ValuLine Bite Registration is an addition-reaction base and catalyst polyvinylsiloxane dental impression material intended to allow dentist to take accurate impressions of patient's occlusal bite registration. ValuLine Bite Registration has a mousse like consistency and high tear strength

INDICATIONS FOR USE

The ValuLine Bite Registration is designed to take an accurate impression of patient's occlusal bite registration.

Part Number	Description	Working Time	Setting Time
[5785-564]	ValuLine Bite Registration Super-Fast Set	30 seconds	1:00 minute
[5785-555]	ValuLine Bite Registration Fast Set	30 seconds	1:30 minutes
[5785-546]	ValuLine Bite Registration Regular Set	1:30 minutes	2:30 minutes
[5785-591]	ValuLine Bite Registration Rigid Fast Set	30 seconds	1:30 minutes
[5785-608]	ValuLine Clear Bite Registration Fast Set	30 seconds	1:30 minutes

BencoDental

ValuLine bite registration

CONTRAINDICATIONS

ValuLine Bite Registration should not be handled with latex gloves. Handling with latex gloves will lead to a slower setting time or the product will not set at all.

MIXING INSTRUCTIONS

1. Insert cartridge into the dispensing gun and remove cap by rotating 90° counter clockwise.
2. Bleed material to ensure both sides are even, then wipe clean and discard.
3. Secure a mixing tip onto the cartridge and extrude a pea size, then apply the material for the desired application.

CAUTION

Federal law in the USA restricts this device to sale only by or on the order of a licensed dentist.

MRI SAFETY INFORMATION

The ValuLine Bite Registration has not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of the ValuLine Bite Registration in the MR environment is unknown. Scanning a patient who has this device may result in patient injury.

STORAGE AND HANDLING

The ValuLine Bite Registration in its original, undamaged packaging is not subject to any special considerations for transport, storage and handling. Store at or below 77°F (25°C)

WARNINGS AND PRECAUTIONS

Product should be inspected for integrity prior to use. Product from damaged packaging should not be used on patients. In the event that the packaging is damaged, the damaged packaging with the product should be returned to the manufacturer and a replacement will be provided only if damage to packaging is caused by product shipment.

Distributed by

BencoDental[™]

800.GO.BENCO • benco.com

295 CenterPoint Blvd., Pittston, PA 18640

IMD50102 Rev A