




**fasTRACT
RETRACTION CAP**

APPLICATION GUIDE


fasTRACT Retraction Cap:

Ideal for use with fasTRACT Retraction Paste

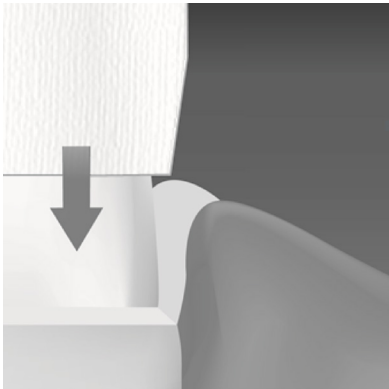

Place fasTRACT
Retraction Paste.

Place fasTRACT Retraction
Cap on tooth structure.

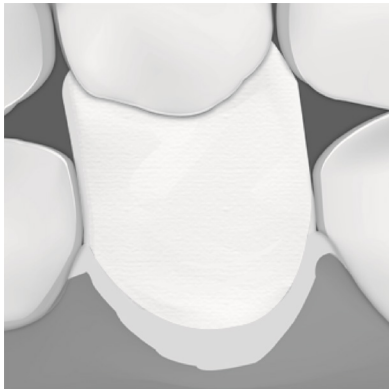

Have the patient bite and
maintain pressure on the cap.
Wait about two minutes,
remove cap, rinse and dry.

Benefits of Using fasTRACT Retraction Caps

- Compression assists with and quickens hemostasis
- Cotton helps with fluid absorption
- Anatomical shape allows easy placement on teeth

Item #	Size
6286-931	#1 BX120
6286-940	#3 BX120
6286-959	#5 BX60

Distributed by

BencoDental

800.GO.BENCO • BENCO.COM
295 CenterPoint Blvd, Pittston, PA 18640