



## **INSTRUCTIONS FOR USE**

### **fasTRACT® Braided Retraction Cord with Epinephrine**

#### **DESCRIPTION**

- 100% cotton braided cord impregnated with stabilized epinephrine.
- Braided construction minimizes fraying and improves handling.
- Activated by gingival fluids to provide hemostasis and retraction.

#### **INDICATIONS FOR USE**

- Impression procedures with elastic materials
- Crown and bridge placement
- Tooth preparation
- Control of sulcular bleeding and fluids

#### **INSTRUCTIONS FOR USE**

- Select appropriate cord size
- Cut length to fit around tooth
- Pack cord gently into sulcus
- Use double-cord technique if needed
- Leave for 1–3 minutes
- Remove before procedure

#### **PRODUCT SIZES**

- 4006-960 – Size 00, 96 in
- 4006-979 – Size 0, 96 in
- 4006-988 – Size 1, 96 in
- 4007-003 – Size 2, 96 in



## **CONTRAINDICATIONS**

- Epinephrine sensitivity or allergy
- Cardiovascular disease where vasoconstrictors are contraindicated
- Uncontrolled hypertension
- Hyperthyroidism
- Use caution in diabetic patients

## **WARNINGS**

- May cause systemic effects if absorbed in significant amounts
- Use caution in medically compromised patients
- Allergic reactions may occur
- Avoid excessive or prolonged use

## **PRECAUTIONS**

- For professional dental use only
- Do not ingest
- Keep out of reach of children
- Use minimal amount necessary
- Limit use to one quadrant per appointment
- Single-use only

## **STORAGE**

- Store in cool, dry place
- Keep container closed
- Avoid heat and moisture

RX ONLY