

Benco Dental™ Orange Solvent

WITH LANOLIN

Product Description

Benco Dental™ Orange Solvent with Lanolin is designed for the removal of zinc-oxide eugenol, rubber base, and other dental impression materials from hands, face, mixing slabs, and instruments. This solvent cleans quickly and easily while remaining non-irritating to the skin.

Indications for Use

For professional dental use in the cleaning and removal of:

- Zinc oxide–eugenol residues
- Rubber base and other impression materials
- Residual material on hands, face, mixing slabs, and instruments

Directions for Use

1. Moisten a clean cloth or swab with Benco Dental™ Orange Solvent.
2. Wipe away impression material from the surface to be cleaned.
3. Discard cloth or swab after use.
4. Close container tightly after each use.

Warnings

- Causes serious eye irritation.
- May cause an allergic skin reaction.
- Do not take internally.
- Contains lanolin — individuals sensitive to lanolin should not use this product.
- For professional dental use only.
- Refer to the product Safety Data Sheet (SDS) for additional health, safety, and disposal information.

First Aid Measures

Eyes: Immediately flush with water for at least 15 minutes. Seek medical attention if irritation persists.

Skin: Wash with soap and water. Discontinue use if irritation or rash occurs.

Ingestion: Do not induce vomiting. Seek medical attention immediately.

Inhalation: Move to fresh air. Seek medical attention if symptoms persist.

Storage & Handling

- Keep container tightly closed when not in use.
- Store in a cool, dry place away from heat and open flame.
- Keep out of reach of children.

Product Information

Net Contents: Available in 8 fl oz (235 mL) and 32 fl oz (946 mL) bottles

REF: [3966-336] and [3966-354]

Country of Origin: USA

Manufacturer

Benco Dental

295 CenterPoint Boulevard

Pittston, PA 18640 USA

1-800-GO-BENCO

benco.com