

CURODONT™ REPAIR FLUORIDE PLUS

CURODONT™ REPAIR FLUORIDE PLUS –
PROFESSIONAL ANTICAVITY RINSE

For Guided Enamel Remineralization of white spots.

- Remineralizes enamel
- Aids in the prevention of dental cavities
- Helps to avoid invasive treatments



10 Applicators CURODONT™ REPAIR FLUORIDE PLUS, article no. P580.USC

[6053-969]

BencoDental™

Made in Switzerland

vVARDIS Whitening Serum

A REVOLUTIONARY WHITENING TECHNOLOGY

The vVARDIS Whitening Serum is a revolutionary whitening treatment that is gentle on teeth and easy to use at home. Its patented WX™ formula, combining the original vVARDIS Technology with Hydroxyapatite, whitens teeth, porcelain crowns and veneers without peroxide and without causing sensitivity.

UNIQUE BENEFITS

- Effective and gentle whitening without attacking the enamel
- Does not cause sensitivity
- Does not cause gum irritation
- Easy and fast application (3 min)
- Visible brightening of restorations, porcelain crowns and veneers
- Suitable also for teenagers
- Vegan, clean, sustainable

HOW TO USE

For instructions on how to use our Whitening Serum, please visit vvardis.com.



vVARDIS
PROFESSIONAL
SWITZERLAND
CREATING TEETH THE WAY NATURE DOES IS THE FUTURE

CURODONT™ REPAIR FLUORIDE PLUS
Anticavity rinse for
the non-invasive
prevention therapy



Guided Enamel Remineralization
to keep your teeth healthy

Healthy teeth – improved quality of life

NEW POSSIBILITIES IN ENAMEL REMINERALIZATION

Remineralizing treatment solutions are now more in demand than ever. Anti caries therapy aims to maintain tooth hard tissue by minimizing the number of restorative interventions. CURODONT™ REPAIR FLUORIDE PLUS enables with the innovative formulation and application a new anticavity treatment.

CURODONT™ REPAIR FLUORIDE PLUS aids to prevent cavities by restoring the enamel in the depth of a white spot.

GUIDED ENAMEL REMINERALIZATION

CURODONT™ REPAIR FLUORIDE PLUS brings fluoride where it belongs – directly to the white spot. With the specially designed applicator, fluoride is guided to where it is most needed.

LOCAL ACTION

The unique CUROLOX® formulation and the application procedure help minerals, like fluoride and calcium found in saliva, to penetrate the enamel surface into the white spot. Lost enamel is remineralized.


“We don’t have to imagine chemistry that literally grows into the tooth, now we have it!,,

Dr. Brian B. Nový, Director of Clinical Innovation, DentaQuest, Westborough, MA, USA

Experts recommend

AN INTELLIGENT TREATMENT CONCEPT

Guided Enamel Remineralization (GER) closes the gap between simple prevention and invasive restorations. Targeting white spots, dentists can now offer their patients a new in-office anticavity therapy. Especially for their youngest patients, CURODONT™ REPAIR FLUORIDE PLUS offers an effective treatment helping to preserve the natural teeth.

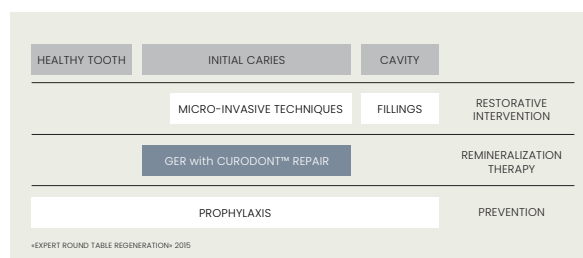

USES

Anticavity dental rinse, for professional in – office treatment of buccal-, occlusal and proximal – white spots. Suitable for patients 4 years and older.

- Prevention of dental cavities
- Remineralizes enamel
- Helps to repair, strengthen and protect teeth

“CURODONT™ REPAIR FLUORIDE PLUS is the classic practice builder. It really motivates my patients to stick to their recall appointments.,,

Dr. Markus Schlee, Forchheim, Germany

Efficiency in daily practice

THE BEST TREATMENT SUPPORT – STRAIGHTFORWARD, DIRECT APPLICATION

Guided Enamel Remineralization builds on the principles of personal dental hygiene and is based on established plaque-biofilm-management. In daily practice, the treatment can be easily integrated into the professional dental hygiene routine. After a short standard preparation of the white spot, CURODONT™ REPAIR FLUORIDE PLUS can effectively be put into action.

- Professional tooth cleaning
- Removal of the pellicle using 2% sodium hypochlorite (20sec.)
- Removal of inorganic deposits using 35% phosphoric acid (20 sec.)
- Rinse tooth surface with water and dry
- Apply CURODONT™ REPAIR FLUORIDE PLUS



Professional tooth cleaning



Cleaning with sodium hypochlorite



Enamel etching with phosphoric acid gel. Unwaxed dental floss is suitable for distributing the gel in the interdental spaces.



Apply CURODONT™ REPAIR FLUORIDE PLUS. The solution spreads around the contact points and diffuses deep into the body of the lesions.

“CURODONT™ REPAIR FLUORIDE PLUS — The first step in life-long integrity of our patient’s teeth.,,

F. Broeseler, DDS, Periodontist, Aachen, Germany