

BencoDental

e-on®

Cavity Conditioner [6209-023]

INSTRUCTIONS FOR USE

Product description:

e-on® Cavity Conditioner is a mild polyacrylic acid solution designed to clean the cavity and to condition dentin and enamel, thus enhancing the bond between glass ionomer cement and tooth structure.

Indications/Intended use:

- Cavity cleaning
- Conditioning of dentin and enamel

Performance features:

The performance features of the product meet the requirements of the intended use.

Contraindications:

Do not use with patients with known allergic reactions to glassionomer cements. In case of an allergic reaction contact a physician.

Patient target group:

Persons who are treated during a dental procedure.

Intended users:

This medical device should only be used by a professionally trained dental practitioner.

Application:

1. After having the tooth prepared apply e-on® Cavity Conditioner to the bonding surfaces for 20 seconds by using a cotton pellet or brush.
2. Rinse thoroughly with water and dry gently. Do not desiccate!
3. Now the glassionomer cement can be applied.
4. Close the bottle immediately after use.

Additional Notes/Warnings:

- Avoid contact with skin, mucous membrane and eyes
- If the material comes into contact with skin, In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Keep away from children!

Storage:

Store e-on® Cavity Conditioner in a cool place at 4-25 °C (39-77 °F). Temperature should not exceed 25 °C (77 °F).

Disposal:

Disposal of the product according to local authority regulations.

Warranty:

Benco Dental warrants this product will be free from defects in material and manufacture. Benco Dental makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Benco Dental's sole obligation shall be repair or replacement of the Benco Dental product.

Limitation of Liability:

Except where prohibited by law, Benco Dental will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.



Distributed by

BencoDental™

800.GO.BENCO • BENCO.COM
295 CenterPoint Blvd, Pittston, PA 18640